



## ÜRÜN ADI

Probe Temizleyici

## ÜRÜNÜN KULLANIM AMACI

Probe Temizleyici Mindray'e ait BC-20, BC-21, BC-10, BC-11, BC-6800, BC6600, BC6000, BC-5000, BC-5150, BC-5390 CRP, BC-5180 CRP, BC-30s, BC-31s, BC-30e, BC-20s, BC-21s, BC-52120, BC-5130, BC-5310 CRP, BC-5100 CRP, BC-5140, BC-30, BC-31, BC-6000Plus, BC-10e, BC-6100, BC-6100Plus, BC-6200, BC6600Plus, BC-6700Plus, BC-6800Plus, CAL6000, tam otomatik hematoloji analizörleri, BC-5000 Vet tam otomatik hematoloji analizörü SC-120 Otomatik Yayma & Boyama Sistemi ve CRP-M100 spesifik protein analizöründe rutin probe bakımı için kullanılabilir.

## ÜRÜN AKTİF İÇERİĞİ

Yüzey aktif madde.....%0.2  
Sodyum hipoklorit.....%10  
Bazik dengeleyeci.....%5  
Koruyucu.....%1

## DİKKAT VE UYARILAR

1. Probe temizleyici aşındırıcı ve tahriş edicidir. Bu ürünü yutmayınız, solumayınız. Yutma veya soluma halinde acilen doktorunuza danışınız. Deriyle temastan kaçınınız, deriyle temas halinde bol miktarda su ile yıkayınız, gerekirse doktorunuza danışınız. Gözle temastan kaçınınız, gözle temas halinde bol su ile yıkayıp derhal doktorunuza danışınız.
2. Son kullanma tarihi geçmiş ürünleri kullanmayınız.
3. Atık ürünleri yerel devlet düzenlemelerine göre bertaraf ediniz.
4. Bu ürün yalnızca nitelikli/eğitilmiş laboratuvar ve sağlık çalışanları tarafından kullanılmalıdır.
5. Bu ürünü kullanırken, uygun kişisel koruyucu ekipman (eldiven, laboratuvar önlüğü, laboratuvar gözlüğü vs.) kullanınız ve güvenli laboratuvar prosedürlerine uyunuz.

## SAKLAMA VE KARARLILIK

Ürün 2°C ~ 30 °C sıcaklıkta karanlık ve kapalı bir yerde, ağzı açılmadığı takdirde, son kullanım tarihine kadar kararlıdır.

Ağzı açılmış olan ürünler 2°C ~ 30 °C sıcaklıkta kullanıldıktan sonra, ağzı sıkıca kapatılıp karanlık ve kapalı bir yerde saklandığı takdirde 60 gün boyunca kararlıdır.

## AMBALAJ ŞEKLİ

4 × 100 ml

## ÜRETİCİ BİLGİLERİ

### Üretici:

BİO AKTİF KİMYA İTH. İHR. İML. SAN. ve TİC. LTD. ŞTİ.

### Adres:

İvedik Organize Sanayi Bölgesi, Güteryüz Sanayi Sitesi, 1352. Cadde, No:24, Yenimahalle/Ankara-Türkiye, ZIP: 06378

**Web:** www.bioaktifkimya.com

**Tel:** +90 312 395 07 81

**Faks:** +90 312 395 07 82

**e-posta:** info@bioaktifkimya.com

## SEMBOLLER



Üretici firma



Uyarı



Kullanım talimatı



30°C

2°C

Saklama sıcaklık sınırı



Son kullanma tarihi



LOT Bilgisi



In vitro tıbbi tanı



Referans numarası



# Probe Cleanser

## NAME OF PRODUCT

Probe Cleanser

## INTENDED USE OF PRODUCT

*Probe Cleanser* is used on Mindray's BC-20, BC-21, BC-10, BC-11, BC-6800, BC6600, BC6000, BC-5000, BC-5150, BC-5390 CRP, BC-5180 CRP, BC-30s, BC-31s, BC-30e, BC-20s, BC-21s, BC-52120, BC-5130, BC-5310 CRP, BC-5100 CRP, BC-5140, BC-30, BC-31, BC-6000Plus, BC-10e, BC-6100, BC-6100Plus, BC-6200, BC6600Plus, BC-6700Plus, BC-6800Plus, CAL6000, fully automatic hematology analysers, BC-5000 Vet fully automatic hematology analyser, SC-120 automated slide stainer & makers and CRP-M100 specific protein analysers routine probe maintenance and cleansing.

## ACTIVE COMPONENTS OF THE PRODUCT

Surfactant.....%0.2  
Sodium hypochlorous.....%10  
Basic stabiliser.....%5  
Preservative.....%1

## WARNINGS AND PRECAUTIONS

1. Probe cleanser contents are corrosive and irritating. Do not swallow or breathe in the vapours of this product, if any accident occurs consult your physician immediately after exposure. Avoid skin contact, if any accident occurs rinse the contaminated area with lots of water and seek medical attention if necessary. Avoid eye contact, if any accident occurs rinse the eye with plenty of water and seek medical attention immediately!
2. Do not use expired products.
3. Dispose of any waste product and canisters according to your local state authorities regulations.
4. This product should only be used by medical professionals or trained laboratory staff.
5. When using this product, use proper protective equipment (such as but not limited to: safety goggles, lab coat, gloves etc.) and abide by safe laboratory procedures.

## STORAGE AND STABILITY

Unopened canisters of this product is stable until its designated expiration date on the label as long as the product is kept in a dark, closed environment under 2°C ~ 30 °C.

Opened canisters of this product is stable for 60 days maximum as long as the product is sealed tightly after use and kept in a dark, closed environment under 2°C ~ 30 °C .

## PACKAGE CONTENTS

4 × 100 ml

## MANUFACTURER INFORMATION

### Manufacturer:

BİO AKTİF KİMYA İTH. İHR. İML. SAN. ve TİC. LTD. ŞTİ.

### Address:

İvedik Organize Sanayi Bölgesi, Güteryüz Sanayi Sitesi, 1352. Cadde, No:24, Yenimahalle/Ankara-Türkiye, ZIP: 06378

**Web:** www.bioaktifkimya.com

**Tel:** +90 312 395 07 81

**Fax:** +90 312 395 07 82

**e-mail:** info@bioaktifkimya.com

## SYMBOLS



Manufacturer



Warning



Instructions of use



30°C

Temperature limitation



Expiration date



LOT

LOT information



IVD

In vitro diagnostics



REF

Reference number